



GUESS WHO
CAME IN
WITH YOUR
PATIENTS! —

YOU DESERVE THE RIGHT PROTECTION

SKYBREEZE*

Powder-Free Nitrile Exam Gloves



A QUALITY GLOVE AT A COMFORTABLE PRICE

Compared to standard nitrile gloves, SKYBREEZE* Powder-free Nitrile Exam Gloves are more cost-effective, lighter, and thinner.

These gloves are a versatile option for various tasks and serve as a great alternative to latex or vinyl exam gloves, providing both comfort and a reliable barrier protection.

The personal protective equipment gloves are approved for use with numerous chemotherapy drugs, for better peace of mind when coming into contact with these potentially hazardous drugs.



DESIGNED TO BE COMFORTABLE, ECONOMICAL, AND PROTECTIVE

- Suitable glove choice for a wide range of tasks
- Tested with various chemotherapy drugs and several commonly used chemicals
- Skin friendly formulations and ISO 10993-10 compliant gloves offer excellent comfort

- Anatomical thin design for enhanced tactile sensitivity
- Viral barrier protection
- Powder-free and Natural rubber latex-free formulation reduces the likelihood of skin irritation and Type I allergies
- Textured fingertips for better grip
- Non-sterile
- SMARTPULL* dispenser facilitates glove dispensing and reduces glove waste by 38.2%¹.

1. O&M Halyard. Study report glove dispensing. GL-DSR00955.

GUESS WHO CAME IN WITH YOUR PATIENTS!

YOU DESERVE THE RIGHT PROTECTION



SKYBREEZE*

Powder-Free Nitrile Exam Gloves



SKYBREEZE* Powder-Free Nitrile Exam Gloves

Item	Description	Size	Packaging
47373	SKYBREEZE* Powder-Free Nitrile Exam Gloves	X-Small	200 items/box 10 boxes/case
47374	SKYBREEZE* Powder-Free Nitrile Exam Gloves	Small	200 items/box 10 boxes/case
47375	SKYBREEZE* Powder-Free Nitrile Exam Gloves	Medium	200 items/box 10 boxes/case
47376	SKYBREEZE* Powder-Free Nitrile Exam Gloves	Large	200 items/box 10 boxes/case
47377	SKYBREEZE* Powder-Free Nitrile Exam Gloves	X-Large	180 items/box 10 boxes/case

Product specifications

Palm width	
X-Small	70 mm
Small	80 mm
Medium	95 mm
Large	110 mm
X-large	120 mm
Gauge thickness measurements	
Middle finger	≥ 0,05 mm
Palm	≥ 0,05 mm
Cuff	≥ 0,04 mm
Average length	240 mm

Quality standards

- Product CE marked as per Regulation (EU) 2017/745 on Medical Devices. Class of the device: I.
- Product CE marked as per Regulation (EU) 2016/425 on Personal Protective Equipment. Class of the equipment: PPE Category III.
- Complies with EN455-1, EN455-2, EN455-3, EN455-4; EN ISO 21420; EN ISO 374-1 Type B glove, EN374 -2, EN374-4, EN16523-1, EN ISO 374-5, ISO 16604.
- AQL of 1.5 for freedom from holes according to EN 455-1.
- Approved for Food Contact as per Regulation 1935/2004 and 10/2011.

Physical properties

Before aging	
Median force at break	6.0N
After aging	
Median force at break	6.0N

Chemotherapy Test Information

The following chemotherapy drugs and concentration had NO breakthrough detected up to 240 minutes:

Tested Chemotherapy Drug And Concentration	Minimum Breakthrough Detection Time (minutes)
Cisplatin (1.0 mg/ml)	No breakthrough up to 240 minutes
Cyclophosphamide (20.0 mg/ml)	No breakthrough up to 240 minutes
Dacarbazine (10.0 mg/ml)	No breakthrough up to 240 minutes
Doxorubicin HCl (2.0 mg/ml)	No breakthrough up to 240 minutes
Etoposide (20.0 mg/ml)	No breakthrough up to 240 minutes
5-Fluorouracil (50.0 mg/ml)	No breakthrough up to 240 minutes
Methotrexate (25.0 mg/ml)	No breakthrough up to 240 minutes
Paclitaxel (6.0 mg/ml)	No breakthrough up to 240 minutes
Vincristine Sulfate (1.0 mg/ml)	No breakthrough up to 240 minutes

Chemotherapy Test Information

The following chemotherapy drugs and concentration showed breakthrough detected in less than 60 minutes:

Carmustine (3.3 mg/ml)	Do Not Use
ThioTEPA (10.0 mg/ml)	Do Not Use

Warning: Not for Use with: Carmustine, Thiotepa

Medical Device Class I
CE 2797
(PPE Cat. III)

O&M Halyard, Inc., 9120 Lockwood Blvd., Mechanicsville, VA 23116
In USA, please call 1-844-425-9273 • halyardhealth.com

ECREPI ArcRoyal Unlimited Company, Virginia Road, Kells, Co Meath, Ireland
O&M Halyard Belgium BV, Berkenlaan 8B, 1831 Machelen (Brab.), Belgium
UK RP • RLD Quality Limited Centenary House, Peninsula Park, Rydon Lane, Exeter, EX2 7XE, United Kingdom

GUESS WHO CAME IN WITH YOUR PATIENTS!

YOU DESERVE THE RIGHT PROTECTION



A GLOVE SOLUTION FOR ALL YOUR PROTECTION NEEDS

From everyday facility wide gloves, to SPECIALTY gloves that offer increased protection against cytotoxic drugs and dangerous chemicals, the HALYARD* Nitrile Exam Glove range provides confidence in protection, regardless of the clinical setting.



HEALTHCARE

HEALTHCARE, NURSING CARE,
DENTISTRY

